

# PRESTO™

Inflation Device

## Inflating to **ultra-high atmospheres**

Ergonomic design allows for comfortable handling

The Only Duo Designed and Rated Up To 40 ATM

Large barrel allows for rapid and easy deflation

40  
ATM



# The only duo designed and rated up to 40 ATM - Presto™ Inflation Device and Conquest™ 40 PTA Dilatation Catheter

**PRESTO™**  
Inflation Device

## Ordering information

Order code	Description	Quantity
ID4030	Presto Inflation Device	5 / box

### Presto™ Inflation Device

#### Indications for use:

The Presto™ Inflation Device is indicated for use with angioplasty balloon dilatation catheters to create and monitor the pressure in the angioplasty balloon dilatation catheter and to deflate the angioplasty balloon dilatation catheter.

#### Contraindications:

None known.

#### Warnings:

**1)** Contents supplied STERILE using ethylene oxide (EO). Non-Pyrogenic. Do not use if sterile barrier is opened or damaged. Single patient use only. Do not reuse, reprocess or re-sterilize. **2)** This device has been designed for single use only. Reusing this medical device bears the risk of cross-patient contamination as medical devices – particularly those with long and small lumina, joints, and/or crevices between components – are difficult or impossible to clean once body fluids or tissues with potential pyrogenic or microbial contamination have had contact with the medical device for an indeterminable period of time. The residue of biological material can promote the contamination of the device with pyrogens or microorganisms which may lead to infectious complications. **3)** Do not resterilize. After resterilization, the sterility of the device is not guaranteed because of an indeterminable degree of potential pyrogenic or microbial contamination which may lead to infectious complications. Cleaning, reprocessing and/or resterilization of the present medical device increases the probability that the device will malfunction due to potential adverse effects on components that are influenced by thermal and/or mechanical changes. **4)** After use, this device may be a potential biohazard. Handle and dispose of in accordance with acceptable medical practices and applicable local, state and federal laws and regulations. **5)** To reduce the potential risk of air embolism, never use air or other gaseous medium to inflate angioplasty balloon dilatation catheters. Ensure all air has been purged from the entire fluid path prior to patient use. **6)** Do not exceed 40 atm when inflating the device. Damage to the device or user injury may result. **7)** Refer to the angioplasty balloon dilatation catheter instructions for use for additional warnings.

#### Precautions:

**1)** Carefully inspect the device prior to use to verify that it has not been damaged during shipment. Do not use if device damage is evident. **2)** Discontinue use of the device if damage, malfunction or contamination is suspected during use. **3)** For experienced physician use only. **4)** Refer to the angioplasty balloon dilatation catheter instructions for use for additional precautions.

### Conquest™ 40 PTA Dilatation Catheter

**Warning: Do not exceed the RBP recommended for this device. Balloon rupture may occur if the RBP rating is exceeded. To prevent over pressurization, use of a pressure monitoring device or indicated syringe is recommended.**

**Please consult product labels and instructions for use, indications, contraindications, hazards, warnings, and precautions.**



BD Switzerland Sarl, Terre Bonne Park – A4, Route De Crassier, 17, 1262 Eysins, Vaud. Switzerland.

[bd.com](http://bd.com)



2797

BD, the BD logo, Conquest and Presto are the property of Becton, Dickinson and Company or its affiliates.  
© 2021 BD. All rights reserved. Bard has joined BD. BD-42940

