

Frequently asked questions

1. Will I feel the suction used with the PureWick™ Urine Collection System?

No, the PureWick™ Urine Collection System is designed so that suction should not be felt. The device is held in place by your anatomy.

2. How often should the PureWick™ Female External Catheter be changed?

The device should be changed at least every 8–12 hours, or if soiled with faeces or blood.

3. Can the PureWick™ Urine Collection System be used when I'm on my side or sitting in a chair?

Yes – the PureWick™ Urine Collection System may be used when lying down or reclining or on your side, as well as when seated in a chair. Care must be taken to ensure that the product is not positioned beneath the tailbone.



PureWick™ Urine Collection System

Now available for home use.
A breakthrough for women with urinary incontinence.



Visit our website to learn more about the PureWick™ Urine Collection System at <https://www.bd.com/en-uk/products-and-solutions/products/product-families/purewick-female-external-catheter>

BD Switzerland Sarl, Terre-Bonne Park - A4,
Route de Crassier 17, 1262 Eysins, Switzerland

For more information on PureWick
visit www.bd.com.

bd.com



What is the PureWick™ System?

The PureWick™ Urine Collection System is designed to better manage your urinary incontinence. It works with the PureWick™ Female External Catheter to wick urine away from the body into a designated collection canister. The PureWick™ Female External Catheter is changed every 8–12 hours and the collection canister containing urine is emptied into the toilet. You can now benefit from the PureWick™ Urine Collection System at home.

Benefits of using the PureWick™ System.



Helps keep the skin dry by wicking away urine.





May help to reduce the risk of nighttime falls, by staying in bed.*



Can make better sleep possible.*

How does the PureWick™ Urine Collection System work?

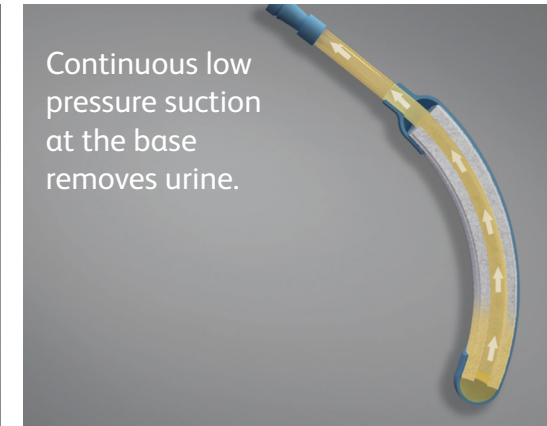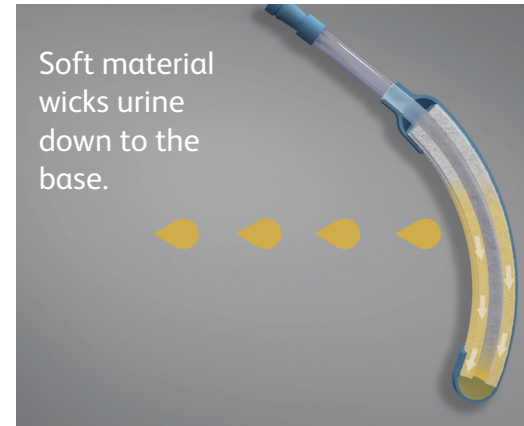

The PureWick™ Female External Catheter connects to the PureWick™ Urine Collection System.



The PureWick™ Female External Catheter is placed between your legs in order to collect urine.

**Based on survey responses (01/19) by 141 caregivers and users who purchased the PureWick™ System and a subsequent supply of PureWick™ Female External Catheters.